

# SealSAFE

Automatic filling system and under-vacuum specimens resealing




Sealsafe is an innovative instrument that allows use of formalin in a safe, controlled and automated way.

- No formalin vapors
- Completely automated
- Optimized fixation
- Improved storage and filing



**1 - Place specimen in the vacuum bag and put it inside the instrument cavity**

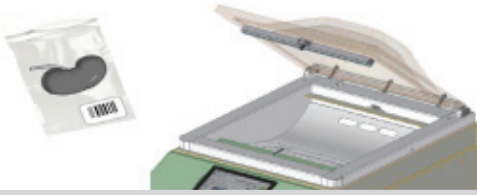


Close the lid of the cavity (you can reseal the bag using it from the operating room through storage).

**2 - Instrument automatically weighs the specimen**



A load cell in the cavity automatically weighs the specimen.

**4 - Sealing the vacuum bag**

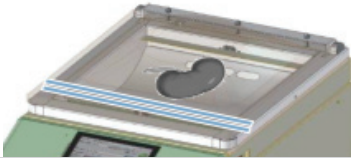

As a final step the SealSafe instrument seals the specimen bag. Fixative injection and sealed are completed in the closed and ventilated cavity.

**3 - The instrument automatically injects the proper volume of fixative into the specimen bag**



The SealSafe automatically dispenses and injects a measured volume of formalin according to the weight of the specimen.

► **Less formalin**

Reduction of volumes thanks to the possibility of decreasing the formalin/sample ratio. Lower disposal costs.

► **Safety**

Full automation prevents exposure of technical staff to formalin. The System is designed to pump formalin directly from commercially available containers.

► **Standardization and Quality**

Consistent injection of the correct volume of formalin according to the weight of the specimen to ensure optimized fixation and histological results.

► **Optimized Storage**

Stand up specimen pouches and optimal preservation. Archival Storage in specimen bags reduces volume by up to 80%.



**MILESTONE**



HELPING PATIENTS

**MILESTONE Srl** - Via Fatebenefratelli, 1/5 - 24010 Sorisole (BG) - Italy  
Tel: +39 035 4128264 - Fax: +39 035 575498  
www.milestonemedsrl.com - email: medical@milestonesrl.com

**UNI EN ISO 9001: 2008 / UNI EN ISO 13485:2004 CERTIFIED**

**MILESTONE MEDICAL TECHNOLOGIES, INC.**  
6475 Technology Avenue, Suite F, Kalamazoo, MI 49009 - USA  
Tel: 269-488-4950 - Toll-free: 866-995-5300 - Fax: 269-488-4949  
www.milestonemed.com - email: info@milestonemed.com

In your country:



CAT349EN-001